

BioCommode

INSTRUCTIONS FOR USE

HYGENEX

Finish

Start

Place the BioCommode into a medical pulp macerator for disposal.

Fold the **BACK LID** down and insert the **FRONT LID** of the BioCommode in the slots. Verify that all pats are firmly embedded.

When the patient has finished, Flip the **BACK LID** to cover the contents. Secure the cover with gentle pressure where the lids meet.


The BioCommode should look like this when it is ready for use.

Place the BioCommode on the patients bottom with the **FRONT LID** supporting their weight. Carefully turn the patient on his back while keeping the BioCommode in place.

If the patient can move, request them to bend their knees and lift their hips

If the patient is unable to move, carefully turn him on his side.

Slide the BioCommode under the patient. Patient's weight rests on the **FRONT LID** with their bottom hovering over the open side on the back.

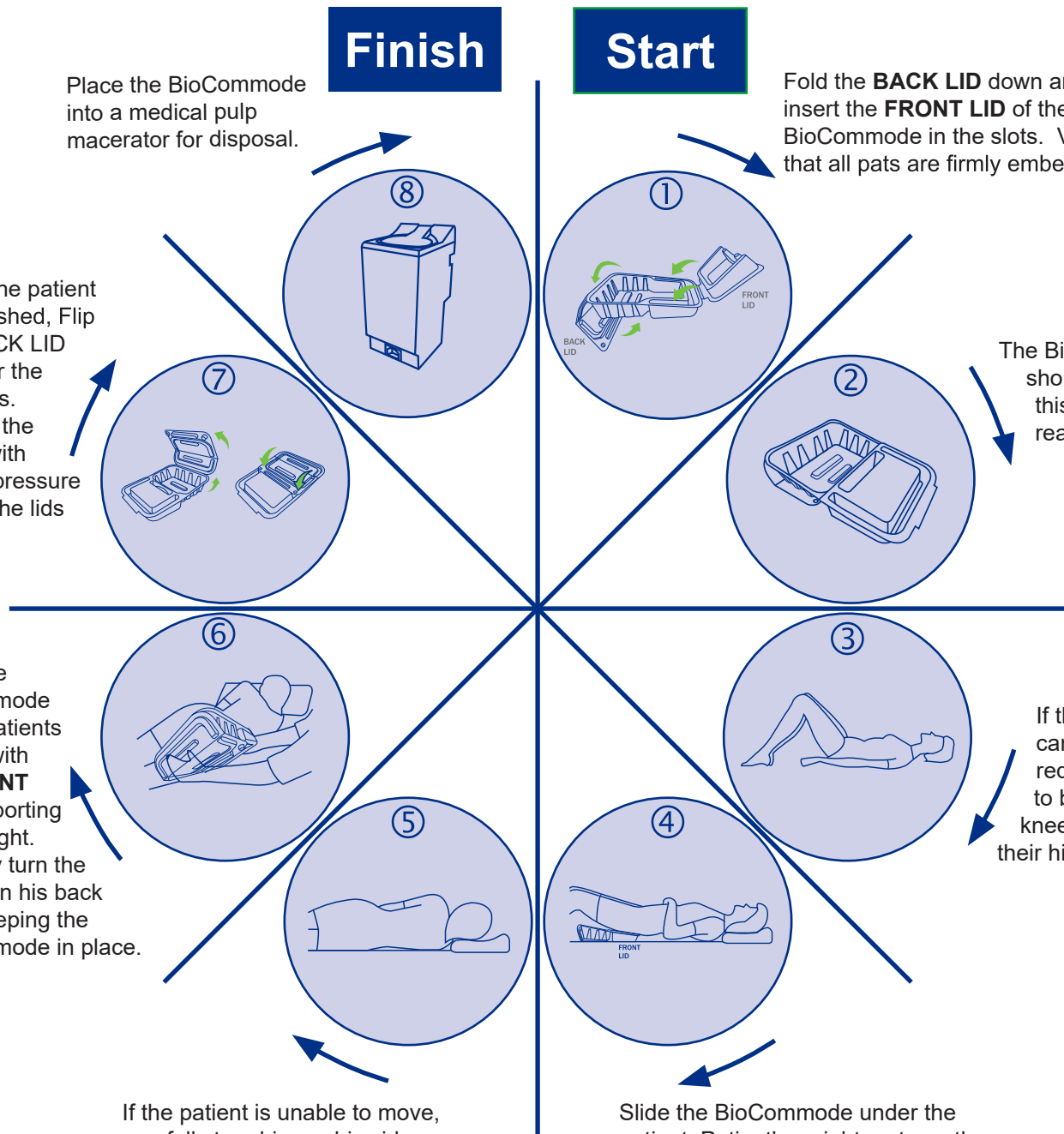

+44 (0) 1202 731555



sales@hygenex.co.uk



www.hygenex.co.uk