

SELINA SUPER SLIPPER PAN

HYGENEX

LONG LIFE REUSABLE SLIPPER PAN SUPPORT

DESIGNED FOR USE IN HOSPITAL WARDS, NURSING AND CARE HOMES, REHABILITATION
AND PERSONAL USE AT HOME



DDC Dolphin offer a large range of products for use in hospital and care home environments.

For more information on our products and services visit our website at ddcdolphin.co.uk

Benefits

- Female specific slipper urinal.
- Ideal for Infant use or during Maternity.
- Lightweight, strong and durable.
- Secure fitting lid to prevent leaks and spills and reduces cross infection and unpleasant odours.
- Can be cleaned in automatic washer disinfectors or hand washed.
- Smooth material surfaces ensure ease of use and ease of cleaning

Specifications

- Female Slipper Urinal with Lid - 1000ml Capacity
- 335mm x 245mm
- Sold individually.
- Weight 0.5kg.
- Registered as a Class 1 Medical Device (CE Marked)
- Complies with following Standards:
 - ISO 9001:2008** Quality Management System
 - ISO 13485:2003** Medical Devices - Quality Management System
 - BS 2588:1995** Reusable Portable Sanitary Pans
- Manufactured from Polypropylene
- Sterilizable up to 134°C

Usage

- For female use.
- For use in a seated position or as a slipper pan under a supine patient.
- Close lid after use to reduce odour and risk of spills.
- Empty after each use.

Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.

Cleaning

- For Hospital Use - Wash before first use. Clean in automatic washer disinfectors compliant to ISO 15883. Wash cycle at 80°C. Disinfection cycle at 90°C. For further processing information, refer to machine manufacturers instructions and local hospital policy.
- For Personal Use - Wash before first use in hot water with detergent. Empty contents after use down a toilet. Rinse pan empty down a toilet. Wash in hot water with detergent after each use and allow to dry naturally.

Product code 90-02-002

Contact us today:



+44(0) 1202 731555



sales@hygenex.co.uk



www.hygenex.co.uk