

Product Range

Skin Cleansing Wet Wipes

Product Type

Flushable, Dispersible & Maceratable Wipes

Distributor

DDC Dolphin Ltd
 The Fulcrum
 Vantage Way
 Poole
 Dorset
 BH12 4NU
 +44 (0)1202 731555
info@ddcdolphin.com
www.ddcdolphin.com



Product Part Code

90-01-089

Country of Manufacture

United Kingdom

Unit of Sale

Product Number	90-01-089
Unit of Sale	Pack / Case / Pallet
Pack Quantity	30 wipes
Pack Dimensions	210 X 110 X 20
Pack Weight (g)	240
EAN Pack Barcode	5060542000406
Case Quantity	48 packs
Case Dimension (mm)	500 x 320 x 260
Case Weight (kg)	19.5
EAN Case Barcode	5060542000444
Wipe Area (cm ² ± 10%)	302
Pallet Dimensions (mm)	1200 x 1000 x 1500
Total loaded pallet weight (kg)	780
Pallet height (mm)	1860
EAN Pallet Barcode	5060542000475
Customs Stat No (HS Code)	34011900

HYGENEX ECO-FRIENDLY SKIN CLEANSING WET WIPES

Product Information

Product Description

Skin cleansing wet wipes consisting of non-woven substrate dosed with aqueous-based liquor, presented in a sealed, laminated, flow wrapped pack with re-sealable opening. 100% biodegradable, flushable, dispersible and maceratable.

Product Features

- A gentle and highly effective skin cleanser which loosens, lifts and locks contamination into the wipe.
- Sting free, rinse free and non-greasy in use.
- Soap, lanolin and alcohol free.
- pH balanced to help maintain the skin's natural protective barrier.
- Dermatologically tested.
- Suitable for sensitive skins.
- Contains premium ingredients in meaningful quantities for exceptional performance.

Intended Uses

- Intended for skin cleansing and is suitable for all areas of the body including intimate areas. Suitable for use on all skin types and ages - from new-borns to the elderly.
- Suitable for post toilet cleansing and intimate hygiene: - Cleanses and lifts faeces, urine and menses from soiled skin.
- Suitable for hands and face cleansing.
- Suitable for nose cleansing particularly when external nasal skin is damaged and sore.
- Suitable for gentle cleansing of greasy, dry, cracked, chafed and chapped skin.
- Suitable for professional use in hospitals, nursing homes, care home and domiciliary care and for personal care in the home or at work.
- Suitable for use on or by patients and for occupational use by care workers to cleanse and care for their skin.
- There is no need to dry the skin after use as this is a leave-on product.
- Suitable for the reduction of skin bioburden by highly effective non biocidal cleansing action.

Nonwoven Material

- 70 gsm 100% cellulosic material
- Disposal - 100% biodegradable, flushable, dispersible, maceratable.
- Low to mid strength material. Generally, non-linting but can lint on facial stubble.
- Product disperses following normal agitation in water during flushing and drain transit. It can be macerated in all macerators and will clear domestic and municipal pumps. It biodegrades under aerobic and anaerobic conditions.
- Cellulosic content derived from sustainable forestry. FSC and PEFC approved.
- Care is required on dispensing due to construction from 100% percent naturally derived material.
- Product meets EDANA/INDA standards for flush ability which require that the following tests are passed.
- Toilet and drain line clearance test (will it flush and transit along drain line).
- Slosh box disintegration test (dispersion).
- Household pump test (anti clog).
- Settling test (anti float).
- Aerobic bio disintegration and bio degradation test.
- Anaerobic bio disintegration and bio degradation test.
- Municipal sewage pump test (anti clog).

HYGENEX ECO-FRIENDLY SKIN CLEANSING WET WIPES

Product Information

Wipes Presentation, Dispensing & Use

- Peel back the anti-rip adhesive opening to access the wipes. This can be temporarily attached in the open position to aid dispensing.
- Wipes are 'Z' folded and non-interleaved for single dispensing.
- Wipe seam is presented in the middle of the aperture for ease of dispensing.
- Wipe seam is always the same direction to aid dispensing. With the pack the correct way around to read the information on it, stroke towards the user with fingertip to elevate edge of wipe. Then pinch lightly between fingertips and gently remove from pack.
- Take care with dispensing cellulosic wipes (flushable/dispersible/maceratable types). The natural materials used in their construction cause a slight adherence between wipes and aggressive removal will cause ripping.
- Take care with removal of the first few wipes as they are packed most tightly in the pack.
- Cleanse the body as required. Allow skin to dry naturally to allow the skin conditioners and moisturisers to work. Do not rinse off. Do not dry unless the patient is cold.
- After defecation, remove gross faeces with toilet tissue and then remove residual faeces with a single wipe.
- Dispose of responsibly.
- Reseal the adhesive opening avoiding creases in the seal which could lead to leakage and the product drying out.
- Use each wipe once only.



DDC Dolphin Ltd

The Fulcrum, Vantage Way, Poole, Dorset BH12 4NU

tel: **01202 731555** | fax: 01202 731666 | info@ddcdolphin.com

DDC Dolphin and Hygenex are Registered Trade Marks