

Slipper Pan Liner and Medical Pulp Support

HYGENEX

INSTRUCTIONS FOR USE

Finish

Start

Carefully roll the patient back to a comfortable position, remove the pulp bedpan from the support and dispose of in a medical pulp macerator.

Position medical pulp bedpan inside bedpan support

The pulp bedpan and support can then be removed from the patient using the underside rim of the support.


Adjust the patients bed to no higher than a 30° angle to provide a comfortable position for toileting.

After toileting, ensure the medical pulp bedpan is secure in the support before gently rolling the patient back onto their side

Carefully roll the patient onto their side.

The patient can then be gently rolled back onto the bedpan. Check with the patient that the bedpan is in a comfortable position prior to toileting.

The pulp bedpan (inside the bedpan support) should be placed against the patients posterior and aligned with the coccyx to ensure comfort.

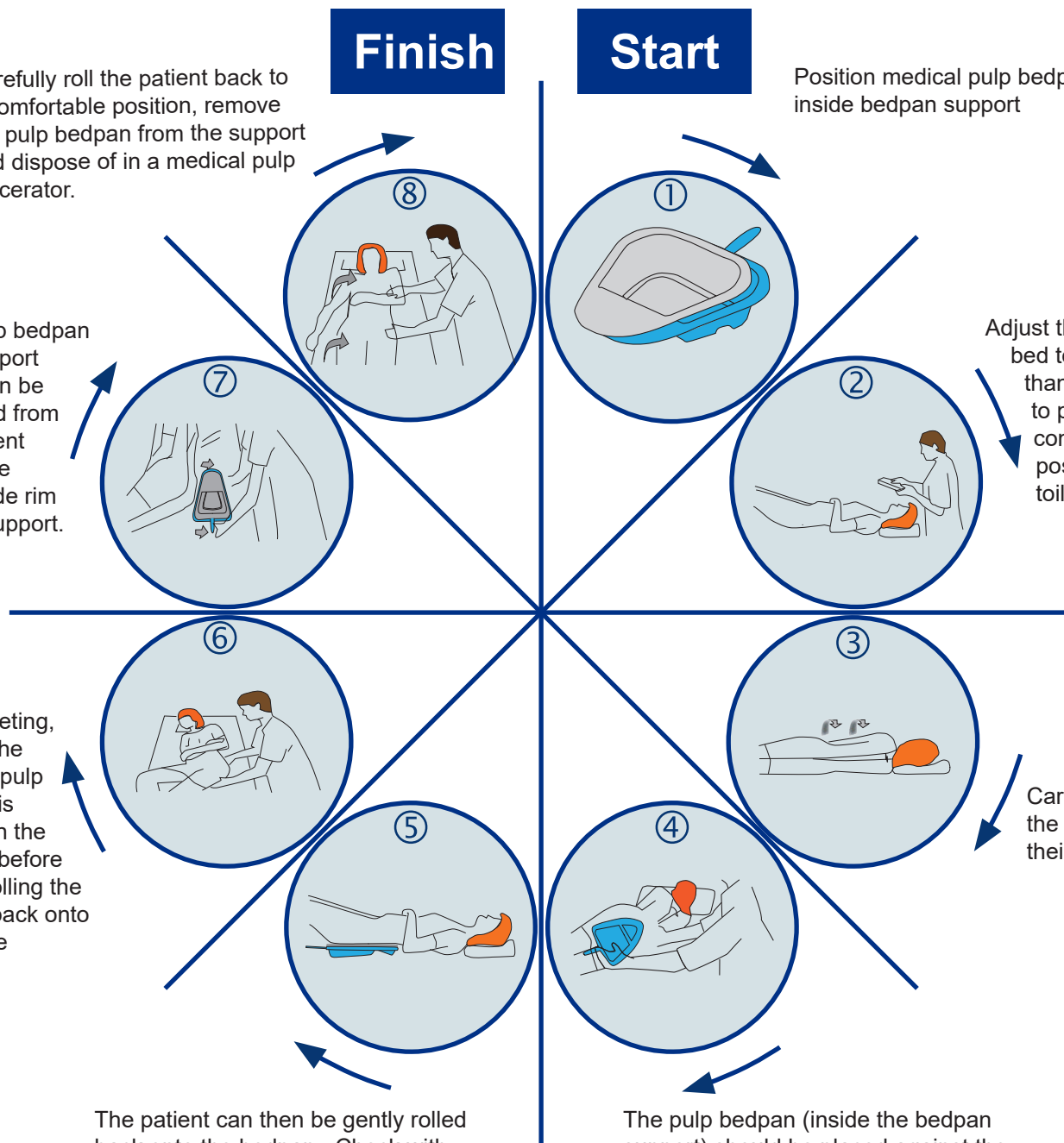

Contact us today :



+44 (0) 1202 731555



sales@hygenex.co.uk



www.hygenex.co.uk